



PRESENTS:

**THE UNIVERSAL MONITRON®
MODEL F0007-B**



The Universal Monitron® provides for a straightforward, independent visual/aural source of respirator/ventilator trending, beyond normal one on one clinical surveillance.

The Monitron® is totally independent of Percussionator® functions, and any display failure within the Monitron® has no effect upon the host device.

The Universal Monitron provides:

- A near instantaneous digital read out of effective cycling frequency.
- Manometric presentation of proximal airway pressure rise and mean.
- Aural alarming for failure to maintain a proximal airway pressure rise.
- An educational means of demonstrating unique programmable functions.

INTERFACING THE UNIVERSAL MONITRON® WITH THE PERCUSSIONAIRE® RESPIRATOR/PERCUSSIONATORS®

Located on the back-side of the Monitron® is a RED service socket which receives proximal airway pressures from the RED service socket of the host device. Special red interfacing tubing is contained in the accessory kit shipped with the Universal Monitron®.

Normally, the Universal Monitron® is installed on the stand post below the host device, using supplied post brackets. Interconnect the Universal Monitron® and the Percussionator® host device red service sockets to servo the Universal Monitron®.


The Manometer is serviced by a Rotary Switch, which can be rotated to INTEGRATED MEAN for a nominal mean proximal airway pressure.

The Frequency Counter can be activated by depressing the “Push to Start” button. Each activation will result in an approximate five minute display period.

The Disconnect Aural Alarm is activated by the manual ON/OFF Switch. Failure to turn the switch off upon purposeful disconnect will result in an aural alarming.

This device is powered by standard internal batteries: AA batteries for the Frequency Counter, and a 9 volt battery for the disconnect alarm.

 Percussionaire® Corporation
1655 Glengary Bay Road, Sagle ID 83860 USA
P.O. Box 817, Sandpoint, ID 83864 USA
Telephone: 208.263.2549
Fax: 208.263.0577

 Percussionaire® Corporation
Belgium Office
21 Broomstraat
1820 Perk, Belgium
Telephone: +.322.751.9134
Fax: +322.751.0923

© Percussionaire® Corp. 2009

www.percussionaire.com

US and Foreign Patents Issued and Pending.

Document ID: F-12002D DCO 10282 06-22-10